

Tubigrip®

Multi-purpose tubular bandage

Tubigrip provides tissue support in the treatment of strains and sprains, soft tissue injuries, general oedema, post- burn scarring and ribcage injuries and is also used for pressure dressings and arm fixation.



About Tubigrip

Tubigrip provides lasting, effective support with complete freedom of movement for the patient. Once the bandage has been applied, covered elastic threads within the fabric move to adjust to the contours of the body and distribute pressure evenly over the surface. Fitting is quick and easy.

Tubigrip stays securely in position without pins or tapes, unlike conventional crepe bandages. As it can easily be reapplied after washing, without loss of effect, Tubigrip is particularly useful for outpatients and in the community.

Tubigrip reduces wastage, as only the exact amount required is cut from the roll. Apply with a 2–3cm overlap.

Tubigrip is available in two shades – natural and beige.

If applied as a double layer to the appropriate limb size, Tubigrip will exert 10–15mmHg to the limb.

Tubular bandage applicators

Tubular Bandage Applicators may be used to make application easier over bulky dressings or painful joints.

Tubigrip is loaded onto the applicator, which is then positioned over the affected area. The end of the bandage is held in position while the applicator is gently removed, leaving the Tubigrip in place. Form a double layer by doubling back the bandage. Ensure that the upper edge of the second layer is taken 2–3cm higher up the limb than the first.

The applicators are available in a full range of sizes and are made from durable chrome-plated steel.

How to apply Tubigrip



1. Cut Tubigrip to twice length required for limb, allowing an extra 2–3cm for overlap.



2. Pull Tubigrip onto limb like a stocking.



3. Double Tubigrip back over limb. Ensure upper edge is taken 2–3cm higher up the limb than the first.

The benefits of using Tubigrip

- Provides comfortable, effective tissue support
- Easy to apply and reapply
- Full range of sizes to suit most applications
- No pins or tapes needed
- Washable and reusable
- Choice of two colours for patient acceptability

Applicators

Applicator Size (Tubigrip Size)	Art. No.
E (B,C)	2913
F (B,C,D)	2915
FX (C,D)	2920
G (D,E,F)	2916
H (E,F,G)	2917
J (F,G)	2918
K (G,J)	2919

Tubigrip Tension Guides are available to help you select the appropriate size and pressure for the condition being treated.

Ordering Details

	Natural Shade			Beige Shade	
	0.5m ⁺ Art. No.	1m Art. No.	10m Art. No.	1m ⁺ Art. No.	10m Art. No.
A Infant feet and arms	–	–	1435	–	–
B Small hands and arms	1510	1520	1436	–	1449
C Medium arms, small ankles	1513	1521	1443	1545	1450
D Large arms, medium ankles, small knees	1511	1522	1437	1546	1451
E Large ankles, medium knees, small thighs	1514	1528	1434	1547	1448
F Large knees, medium thighs	1512	1523	1438	1548	1452
G Large thighs	1515	1524	1439	–	1453
J Large trunks	–	–	1440	–	–
K X-Large trunks	–	–	1441	–	–
L XX-Large trunks	–	–	1442	–	–

