

Avance® Solo Adapt – Fight pressure with pressure

Avance® Solo Adapt Negative Pressure Wound Therapy (NPWT) system is indicated for use on pressure ulcers, where it aims at promoting faster healing, improved patient mobility and a reduced risk of infections.

1. Pump

- Regulates pressure to deliver a continuous -125mmHg negative pressure for 14 days¹.
- Visual and audible alarms for leakage, blockage and low battery, promptly address issues^{1,2}.
- Portable and light weight – minimising impact of patient mobility^{1,3}.

2. 50ml canister

Canister may be replaced by clinician, lay carer or the patient themselves^{4,5}.

3. CFM Technology™


Controlled air inlet ensures uncompromised continuous negative pressure is delivered to the wound bed^{6,7,8}.

4. Avance® Solo Adapt Foam

- A single use polyurethane foam wound filler which can be cut to the shape of the wound.
- Transports exudate via Avance® Solo Adapt Transfer Port.

5. Avance® Solo Adapt Film with Safetac® technology

A conformable film, Safetac soft silicone adhesive produces an effective seal for negative pressure therapy^{9,10}.

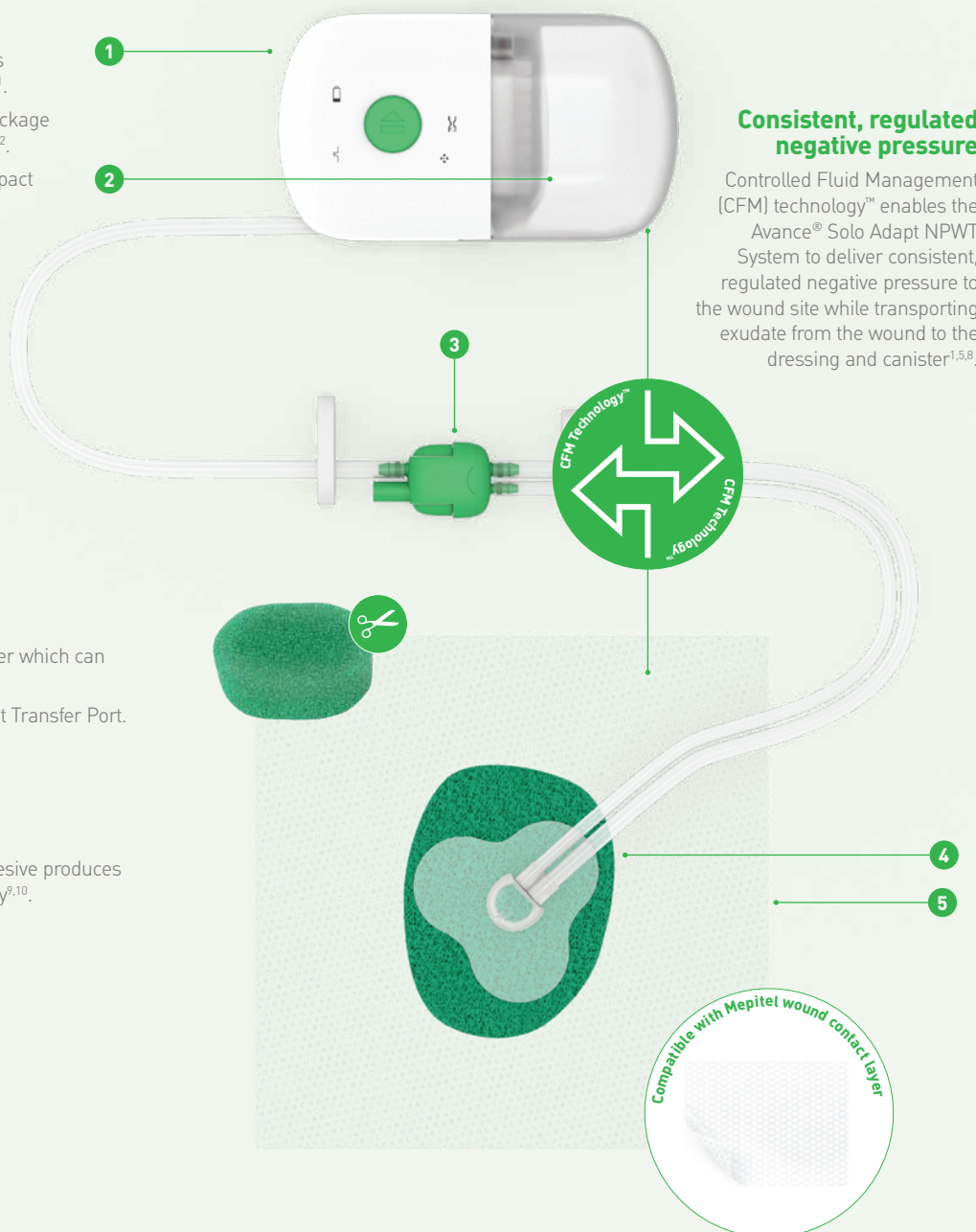

Avance® Solo Adapt

Single-use negative pressure therapy
– without compromise


Mölnlycke®

What is negative pressure wound therapy?

Negative Pressure Wound Therapy (NPWT) is a method of treating wounds using suction. The dressing is airtight and the pump applies negative pressure, clearing the wound of exudate and fluids.

What are the benefits of Avance® Solo Adapt?

- CFM Technology™ – a combination of controlled air flow and a distal canister that enables the Avance® Solo Adapt NPWT system to deliver continuous regulated negative pressure to the wound site, while transporting exudate from the wound to the distal canister^{6,7,8}.
- The pump will deliver therapy for 14 days¹.
- Kit includes a customisable 3cm foam which can be used in conjunction with Mepitel® wound contact layer, to avoid potential for the foam to stick to tissue.
- The visual signals and audible alarms of the pump help to ensure that loss of therapy or system issues are promptly addressed to enable therapy to continue^{1,2}.

Regular pulsing sound

Note that the Avance Solo Adapt system will emit a regular pulsing sound from the cycling of the pump. This is normal and confirmation that the system is operating^{2,5,9}.

Areas of use



dNPWT – Pressure ulcers

Avance Solo Adapt is an NPWT system indicated for use on pressure ulcers, where it is desirable/necessary to be able to customise the dressing to fit to contoured anatomy in order to achieve a seal.

Precautions

You should consult your healthcare professional if you experience any problems. This includes bleeding, elevated blood pressure or issues with the device. For warnings and precautions see the healthcare professional IFU, included in the product package.

Avance Solo Adapt assortment

Code	Description	Individual Sterile Unit Contents
9994152	Avance Solo Adapt 'Starter Set' - 10 x 35cm	Pump + Canister 50ml with tubing + 4 x AA batteries + Film dressing + Foam wound filler
994152	Avance Solo Adapt Film Dressing Kit	Film dressing + Foam wound filler + Transfer port and tubing
994000	Avance Solo Adapt Film Dressing	x 2 Safetac adhesive film dressings

References: 1. Avance Solo Adapt CMM Data on file (ref 16). 2. Avance Solo Adapt CMM Data on file (ref 17). 3. Avance Solo Adapt CMM Data on file (ref 21). 4. Avance Solo Adapt CMM Data on file (ref 14). 5. Avance Solo Adapt CMM Data on file (ref 15). 6. Avance Solo Adapt CMM Data on file (ref 10). 7. Avance Solo Adapt CMM Data on file (ref 18). 8. Avance Solo Adapt CMM Data on file (ref 13). 9. Avance Solo Adapt CMM Data on file (ref 23). 10. Avance Solo Adapt CMM Data on file (ref 22).



The Avance® Solo Border Dressing is available in a wide range of shapes and sizes. **Scan the QR code to find the assortment available in your region. www.AvanceSolo.com**

Find out more at www.molnlycke.co.uk

Mölnlycke, Unity House, Medlock Street, Oldham, OL1 3HS. Tel +44 (0) 800 731 1876
The Mölnlycke, Avance and Safetac trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies. © 2022 Mölnlycke Health Care. All rights reserved. UKWC1098


Mölnlycke®