

# Aldress®


## Multi-layered, absorbent, semi-permeable cover dressing

### Polyester nonwoven

- For stability and ease of application
- Maintains dressing integrity

### Semi-permeable, waterproof film

- Moisture vapour permeable
- Showerproof
- Reduces friction

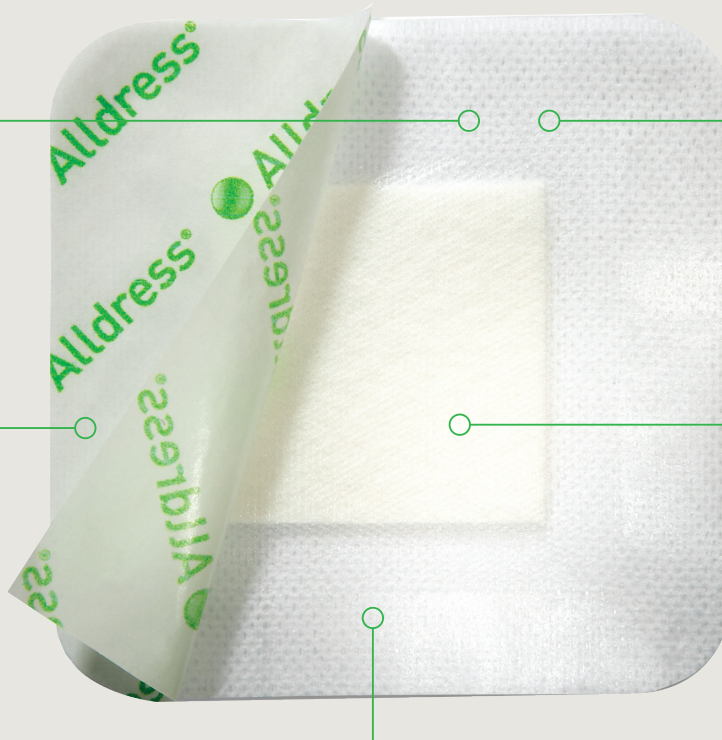

### Polyacrylate adhesive border

- For secure fixation and leakage control
- Skin friendly, water-based, 'latex and solvent free' adhesive

### Highly absorbent, multi-layered wound pad

- Absorbs and retains exudate
- Reduces the risk of maceration
- Discourages lateral wicking


### Specially designed polyester net wound contact layer

- Minimizes adherence to the wound bed
- Permeable to exudate
- Discourages absorption of hydrogels
- Reduces risk of maceration


- All-in-one absorbent, self-adherent, composite cover dressing with a gentle, skin-friendly adhesive for optimal performance and comfort

- Bacterial proof, viral proof and waterproof to protect the patient and the clinician.


## Directions for use

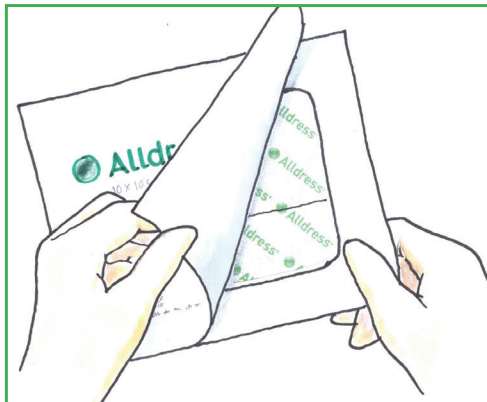

Gently cleanse the wound area and dry surrounding skin. Remove from package; peel away half of the backing paper.

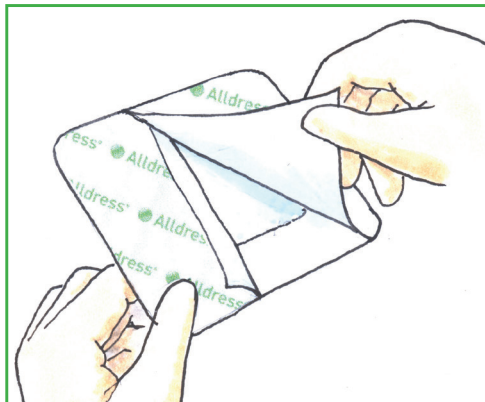

Place the exposed adhesive on the skin next to the wound.

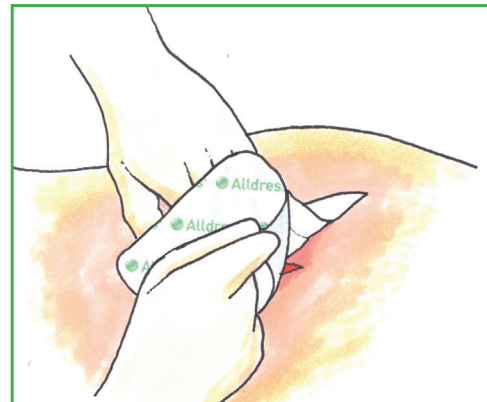

Remove the remaining backing paper and lay dressing over the wound. **DO NOT STRETCH.** Gently smooth the edges in place to ensure secure adhesion.

## How Alldress works

Alldress incorporates a low-adherent wound contact net. The absorbent multilayered pad absorbs and retains exudate which minimizes the risk of maceration.

The skin-friendly, water-based adhesive provides secure fixation and leakage control.

The film backing is bacterial proof, viral proof and waterproof and offers advanced fluid handling promoting optimal moisture balance.

## Benefits of Alldress

- Low adherent wound contact net minimizes the risk of adherence to the wound
- Maintains a moist wound healing environment
- Minimizes the risk of maceration
- Showerproof
- Viral, bacterial and waterproof to protect the patient and the clinician
- Easy to remove protection paper allows for convenient and aseptic application
- Skin-friendly, water-based adhesive
- Highly absorbent

## Indications\*

Alldress is intended to absorb, protect and maintain a moist environment in open and closed wounds. Alldress is intended for both dry and exuding wounds and as a secondary dressing together with, for example, gels and alginates.

## Precautions\*

Alldress should not be used on infected wounds without consulting a health care professional. If reused, performance of the product may deteriorate. Cross contamination may occur. Sterile. Do not use if inner package is damaged or opened prior to use. Do not re-sterilize.

## Alldress® Assortment

| Art. no | Size cm | Pcs/Box | Pcs/Case |
|---------|---------|---------|----------|
| 265320  | 10 x 10 | 10      | 70       |
| 265340  | 15 x 15 | 10      | 100      |
| 265360  | 15 x 20 | 10      | 120      |

\*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to [www.molnlycke.ca](http://www.molnlycke.ca)

